

Body Armor® Pro Term Foot stump orthosis

Interim treatment following Chopart or Lisfranc amputations

WOUNDCARE-
CIRCLE



Indications

- > Diabetic foot syndrome
- > PAOD (peripheral arterial occlusive disease)
- > Trauma

Postoperatively for treatment following

- > Chopart amputation
- > Lisfranc amputation



DARCO

The Body Armor® Pro Term has adopted many useful and sophisticated properties of proven DARCO products and thus even as a product innovation, it is already a typical DARCO product: sophisticated – patient-friendly – cost-effective



wide opening for easy foot insertion



wedge-shaped sole system for optimal rear weight-bearing



practical closure system



perfect fit through air compression



easy inflation and deflation using an integrated system

Body Armor® Pro Term

The interim treatment of patients who have undergone minor amputations always represents a challenge for the technician or physician. For this reason, our development department designed an appropriate, ready-made treatment option. The new foot stump orthosis from DARCO represents interim treatment to be worn for approximately 10 weeks following an amputation. The Body Armor® Pro Term offers a treatment option to allow the foot stump to completely heal during this time. Thereafter a conventional, custom-made shoe is worn.

The wedge-shaped outsole and insole in dorsal extension ensure rear weight-bearing. The stable, ankle-spanning shaft stabilizes the upper and lower ankle and prevents deformation of the stump. The blue inlay with integrated pump compresses and thus maintains the shape of the stump.

The Body Armor® Pro Term represents conservative treatment which can be changed manually, however, and thus be individually customized to patient needs.

ART. NO.	COLOR	SIZE	STUMP LENGTH
BAPT2	Black	M	16 cm
BAPT3	Black	L	19 cm

right and left fit